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Reg. No. at IRAN Medical equipment department: 1855

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Carboxylate Cement

Compositions:

powder contains: Zinc oxide, Magnesium oxide

liquid contains: Polyacrylic Acid.

This product is available in two models:

Model 1: 35 gr Powder and 15 ml Liquid contains

Model 2: 100 gr Powder and 40 ml Liquid contains

Indications:

- ▶ Lining for all filling materials (amalgam, composites).
- ▶ Cementation of inlays, onlays, crowns and bridges made of precious metal, non-precious metal and ceramic.
- ▶ Cementation of orthodontic bands.
- ▶ Cementation of retention pins and screws.
- ▶ Core build-ups.
- ▶ Long-term temporary fillings.
- ▶ Fillings in deciduous teeth.

Advantages:

- ▶ No acid pain
- ▶ For sensitive tooth stumps
- ▶ Easy to use
- ▶ No etching, priming, bonding
- ▶ Compressive strength well above the material standard
- ▶ Cement according to ISO 9917-1

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with **Carboxylate**
- ▶ Use eye protection, mask and gloves when using **Carboxylate**
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only
- ▶ The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.
- ▶ The cement must be applied to the dry tooth surface and must set without contact to moisture.
- ▶ Remove excess material from neighboring teeth, facets etc. immediately. Do not expose liquid to sunlight and frost.
- ▶ Close containers immediately after removing material.
- ▶ Clean instruments and glass plate with water before the cement sets.
- ▶ Zinc polycarboxylate cement can be dissolved using a cement-solvent after setting.

- ▶ Inhalation: May cause irritation to mucous membranes. Move the exposed person to fresh air.
- ▶ Eye contact: Immediately flush opened eye under running water for several minutes, consult an eye specialist.
- ▶ Skin contact: Remove by washing with soap under the running water. Generally the product is not irritating to skin.
- ▶ Ingestion: (A plenty of substance) induce vomiting and consult a physician.

Warning:

- ▶ Keep in a cool, dry, well-ventilated area.
- ▶ Keep containers tightly closed.
- ▶ Do not use after expiration date.

Dispensing Instructions:

- ▶ Cavity lining consistency: 3.6 g powder, 1 g liquid
- ▶ Cementation consistency: 2.9 g powder, 1 g liquid

Instructions for use:

Divide the measured quantity of powder into 4 portions (1/2 1/4 1/8 1/8). Beginning with the largest quantity, stroke the powder into the measured liquid within 60 seconds. The mixture should appear thicker as usual for zinc phosphate cement, yet it is just as easy to work. The cavity lining consistency has been reached when on lifting the spatula, the peak drawn out forms a small hook and does not sink back into the paste. The cementation consistency has been reached when the peak formed on lifting the spatula slowly falls back into the paste. Discard remaining powder or impure liquid.

Mixing time: 60 seconds

Processing Time from beginning of mixing (at 23 °C):

- ▶ For Cementation: 2 minutes
- ▶ For cavity linings: 1:40 minute

Setting Time from end of mixing:

- ▶ For Cementation: 3-4 minutes
- ▶ For cavity linings: 2-3 minute

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

Consult instructions for use

Indicates the batch code

Catalogue number

Indicates the date after which the medical device must not be used

Indicates a medical device that needs production/high source.

Do not use if Package is Damaged

Indicates the medical device manufacturer (Name & Address) process.

Indicates a medical device that has not been subjected to a sterilization process.

Indicates the medical device intended for use by a professional personnel only.

Indicates a medical device that needs to be protected from moisture.

Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

Carboxylate Cement

- ISO13485 : 2016 , US cert, IAS
- ISO 8001: 2007, Tuv certification
- ISO 9001: 2015, Tuv certification
- ISO 10002: 2014, Tuv certification
- ISO 14001: 2015, Tuv certification
- ISO 10004: 2012, Tuv certification

manufacturer

Pardis Morvabon Pharmed Co.,
Head Office: Unit 154, 7nd floor, 7 block,
Bojrsaz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1838
Fax: 0098 21 6690 8819
Factory: Unit 135, 136, 139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran
Tel: 0098 21 7625 1702
Fax: 0098 21 7648 1457

Reg. No. at IRAN Medical
equipment department: 1855