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Chloroform

Extra pure, Stabilized

Most efficient solvent for gutta-percha and varnish removal

Compositions:

Pure Chloroform, stabilizer

advantages?

intended use:

- ▶ Chloroform is supplied as a general solvent that can be used for cleaning up certain hardened cements and materials from tools and mixing equipment. Solvent for gutta-percha and varnish removal.

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Chloroform.
- ▶ Use eye protection, mask and gloves when using Chloroform
- ▶ Avoid contact with oral soft tissue, eyes or skin
If accidental contact with soft tissues, eyes or skin occurs,
- ▶ immediately flush with plenty of water
In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only.
- ▶ Do not use in patients with allergies.
- ▶ It is best to use a cotton swab dipped in this product and not spill into the mouth.
- ▶ At the end of the work, make sure that the rest of the product comes out of the patient's mouth.
- ▶ Avoid using in similar cases.
- ▶ It can be deadly if swallowed.

Its steam is harmful. Avoid breathing steam. This product is not intended for human use or contact.

This product causes cancer and birth defects or birth defects and more.

warning:

- ▶ Do not use after expiration date
- ▶ Keep out of reach of children
- ▶ Store in the original package.

Directon to use?:

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.	 15°C Indicates the Temperature Limit
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected for a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and requires lubrication or oiling of the skin.	 Indicates that the product has a low potential of metal cation or skin corrosion.	

Chloroform

Extra pure, Stabilized

- ISO13485 : 2016 , US cert, IAS, USA
- Iso 9001: 2015, Tuv certification, Germany
- Iso 14001: 2015, Tuv certification, Germany
- Iso 18001: 2007, Tuv certification, Germany
- Iso 10002: 2014, Tuv certification, Germany
- Iso 10004: 2012, Tuv certification, Germany

manufacturer
Pardis Morvabon Pharmed Co.:
Head Office: Unit 154, 7nd floor, 7 block,
Borj saz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1438
Fax: 0098 21 6690 8819
Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran
Tel: 0098 21 7625 1702
Fax: 0098 21 7648 1457