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Alum Stop

Aluminum Chloride %25

Colorless gel for bleeding control and sulcular fluid.

Ingredient: Aluminum chloride, Gel base

Application:

- ▶ Perfect transparency for better performance without staining or staining.
- ▶ Ideal viscosity for spreading without drop.
- ▶ Without interacting with bonding agent.
- ▶ Control of bleeding and gum secretions during gingival close restorations.
- ▶ Gum Solecules Control During Molding.
- ▶ Stop the internal oral surface bleeding with a special application for the front teeth.
- ▶ Transparent gel for anterior restorations.
- ▶ Suoclaric fluid control for better bonding in adhering indirect restorations, Laminate, veneer, Inlay, Onlay.

Intended Use:

- ▶ Prepare the desired surface by washing with water and mildly drying by air pooper.
- ▶ A sufficient amount of gel on the bleeding tissue for 1-3 minutes (to prevent infection, it is better to transfer some gel with a disposable brush to the desired place).
- ▶ The surface of the gel is soaked so that the gel is completely cleaned, washed, then the texture is completely dry.
- ▶ Air pointer should be used mildly.

Precautions:

- ▶ Carefully read and understand instructions before using. It's necessary to use rubber-dam while working with **Alum stop**
- ▶ Use eye protection, mask and gloves when using **Alum stop**
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only

- ▶ Do not use in similar cases.
- ▶ If you use the 7th bond (self-etching bonds), before using the bond, you should clean the surface of the tooth with Pumice paste and then bond. In the case of the 5th generation bond and the use of acid etching does not require this.
- ▶ The product may irritate the mucous membrane of the mouth. Rinse immediately with water in the ringer.
- ▶ In patients with an allergy to Aluminum chloride or chemicals are not used.
- ▶ Before applying any other chemicals, the tissue should be thoroughly washed with water.
- ▶ At the end of the work, the remainder of the material must be completely removed from the patient's mouth.

Warning:

- ▶ Do not use after expiration date
- ▶ Keep out of reach of children
- ▶ Store in the original container at room temperature.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device call require cleaning and reconditioning, or avoidance of staining of the skin	 Indicates the Temperature Limit

Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

Alum Stop

Aluminum Chloride %25 Gel

- ISO13485 : 2016 , US cert, IAS,
- Iso 9001: 2015, Tuv certification,
- Iso 14001: 2015, Tuv certification,
- Iso 18001: 2007, Tuv certification,
- Iso 10002: 2014, Tuv certification,
- Iso 10004: 2012, Tuv certification,

Production License No.
89586041
IMD: 91246235

ISO 13485: 2016

 manufacturer

Pardis Morvabon Pharmed Co.:
Head Office: Unit 154, 7nd floor, 7 block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1438
Fax: 0098 21 6690 8819
Factory: Unit 135, 136, 139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran
Tel: 0098 21 7625 1702
Fax: 0098 21 7648 1457