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EUGENOL (USP GRADE)

Morvabon **Eugenol** is an antiseptic, analgesic and sedative essential oil. When mixed with Zinc Oxide, it is specially recommended for many cases

Composition:

Eugenol Oil

Intended Use:

- ▶ Treatment of exposed pulp and for the disinfection of the root canal.
- ▶ Temporary obturation of the root-canal where pulp is partially exposed (pulp capping)
- ▶ Antisepsis of the root-canal
- ▶ Treatment of Dry Socket that may develop after extraction

Direction for Use:

Temporary obturation:

- ▶ Mix a few drops of Eugenol with Zinc Oxide powder on a glass mixing slab.
- ▶ Spatulate to form a thick paste
- ▶ Prepare the cavity and dry it thoroughly
- ▶ Obturate in the usual manner

Antisepsis of the root-canal:

- ▶ Clean the root canal thoroughly
- ▶ Insert one or two drops of Eugenol with a plastic pipette.
- ▶ Remove any excess liquid with a cotton pellet.

Precautions:


- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with **Eugenol**.
- ▶ Use eye protection, mask and gloves when using **Eugenol**
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water.
- ▶ For dental use only
- ▶ Eye contact – flush with copious amount of water.
- ▶ Seek medical attention if irritation persists.
- ▶ Ingestion–Drink plenty of water and refer to medical centers.
- ▶ Skin contact – wash with soap and water. Seek medical attention if irritation persists.
- ▶ It is not recommended for patients with allergy to eugenol oil.

Warning:

- ▶ Protect from direct sunlight.
- ▶ Keep out of reach of children.
- ▶ Do not use after the expiration date.
- ▶ Store in the original container at room temperature.
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

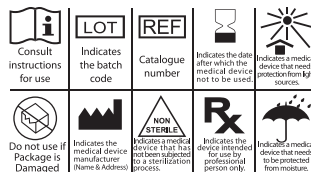Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

EUGENOL (USP GRADE)

- ISO 13485
- ISO 9001-2015
- ISO 14001-2015
- ISO 18001-2007
- ISO 10002-2014
- ISO 10004-2012

Production
License No.
76139023
IMD: 12613988

manufacturer

Pardis Morvabon Pharmed Co.
Head Office: Unit 154, 7nd floor, 7 block,
Borjssaz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1438
Fax: 0098 21 6690 8819
Factory: Unit 135, 136, 139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran
Tel: 0098 21
Fax: 0098 21