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Sodium hypochlorite Solution %5.25

### Liquid for rinsing root canal

**Composition:** sodium hypochlorite (%5,25 of active chlorine), water Ph.Eur. ,stabilizer.

### Intended use:

- ▶ Rinsing the root canal
- ▶ Opening the dentine tubules before restorations
- ▶ Smear layers and necrotic tissues removal
- ▶ Rinsing canals with inflammation periapical of tissues
- ▶ Simultaneous use with EDTA cream (MORVA-PREP) for release oxygen bubbles in canal for cleaning and widening of root canal

### Advantages:

- ▶ Efficient and significant reduction of microorganisms
- ▶ Destroying of Candida albicans colonies
- ▶ Destruction of Entrococcus faecalis microbial biofilm and removal of old biofilm
- ▶ Dissolves the organic matter present in the root canal
- ▶ During mechanical canal widening it removes the non-vital pulp debris
- ▶ It prevents from the teeth discoloration that might occur after filling
- ▶ Significantly effect on positive-gram bacteria rather than chlorhexidine
- ▶ Sodium Hypochlorite %5.25 can be dissolved necrotic tissue and abscess and can be disinfected deep layers

### Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ Re-cap, disinfect and wipe syringe with an intermediate level disinfectant between uses.
- ▶ It's necessary to use rubber-dam while working with Hypo-Endox
- ▶ Use eye protection, mask and gloves when using Hypo-Endox
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.

- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only
- ▶ Dispose of needle tip and empty Luer lock syringe properly.
- ▶ Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check.
- ▶ Use only recommended tips. To avoid cross contamination do not reuse tips.
- ▶ Do not induce vomiting.

### Warning:

- ▶ Do not use after expiration date
- ▶ Keep out of reach of children
- ▶ Protect the product from the sunlight
- ▶ Store in the original containers at temperature 15–25 °C (in refrigerator or room temperature).
- ▶ Once container is open, the expiry date doesn't change if it is securely closed after each use.

### Direction of Use:

Insert the liquid into the root canal by means of the syringe with a needle with side portal. Rinse the canal copiously, approximately 10 ml for each canal. Sodium hypochlorite contained in the product inactivates immediately in contact with organic substance. While concentration of sodium hypochlorite is decreasing, time of rinsing and volume should be necessarily simultaneously increased.

### Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance.

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device must not be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.	 Indicates the Temperature Limit
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the product has a chemical composition capable of being classified as non-flammable.	 Indicates the presence of oxidizing substances.	

Hypo-EndOX %5.25

- ISO13485 : 2016 , US cert, IAS
- Iso 9001: 2015, Tuv certification
- Iso 14001: 2015, Tuv certification
- Iso 18001: 2007, Tuv certification
- Iso 10002: 2014, Tuv certification
- Iso 10004: 2012, Tuv certification

  
MEDICAL EQUIPMENT DEPARTMENT  
Production License No.

 manufacturer  
Pardis Morvabon Pharmed Co.:  
Head Office: Unit 154, 7nd floor,7 block,  
Bohsaz Bd, Azadi Street, Tehran, Province, Iran  
Tel: 0098 21 6658 1838  
Fax: 0098 21 6690 8819  
Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,  
20thkm of Damavand Rd.,, Tehran, Iran  
Tel: 0098 21  
Fax: 0098 21