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## CAVITY VARNISH

### Cavity Varnish based on natural resin

Morvabon Cavity Varnish is a solution of natural resin in an antiseptic solvent and provides an inexpensive and convenient way to seal dentin tubular under cement and restoratives. It forms a protective barrier that prevents the penetration of toxic substances from such materials before they fully cure and contributes to eliminating or reducing post-operative discomfort as well as the occurrence of secondary decays.

#### Composition:

- ▶ Natural resin, Alcohol solvents

#### Intended use:

- ▶ Disinfecting cavity protection varnish for the avoidance of post-operative sensitivity
- ▶ Cavity protection varnish before cement fillings
- ▶ Sealing of dentinal tubules
- ▶ Lining of cavities before amalgam fillings and restorations made of gold to protect against corrosion products
- ▶ Insulation against thermal shock
- ▶ Protective varnish for the surfaces of cement fillings and plaster models

#### Advantages:

- ▶ Ready-to-use solution
- ▶ Made of natural raw materials
- ▶ Contains no chloroform or ethyl ether and other harmful solvent

#### Precaution:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Cavity-Varnish.
- ▶ Use eye protection, mask and gloves when using Cavity-Varnish
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only
- ▶ Eye (contact): Open the eye wide and thoroughly wash for 15 minutes with running water. Seek medical attention.
- ▶ Ingestion: Rinse mouth with water. Gargle with salt water and drink lots of milk. Seek medical attention if symptoms persist.
- ▶ Inhalation: Remove to fresh air. If breathing is difficult, seek medical attention.

#### Warning:


- ▶ Do not use Cavity Varnish with resin-based cement or filling materials as Cavity varnish may interfere with hardening process.
- ▶ In order to prevent damage to the protective varnish film on the surface of cement, the patient must not eat for 1 hour after application.
- ▶ Tightly close the varnish bottle immediately after use to avoid loss of solvent.
- ▶ Avoid contact with open flame.
- ▶ Store at 15-25°C (room temperature).
- ▶ Do not use after expiration date
- ▶ Keep out of reach of children

#### Instructions for use:

Apply a thin layer of resin with a pen or brush in the desired location and wait for the solvent to evaporate and form a thin film layer. Air flow can also be used to speed drying. For better flooding, several layers can be applied repeatedly. Be sure to apply each layer separately and apply after the next layer dries. Avoid applying thick layers and films at once.

#### Contraindication:

This product is not contraindicated for children, pregnant women, the elderly and certain diseases and can be used for all people except those who are allergic to the raw materials of this product (natural resins and alcohol).

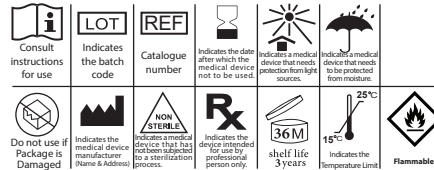Produced by Morvabon Co.  
www.morvabon.com  
info@morvabon.com  
Made in IRAN

CAVITY VARNISH

manufacturer

Pardis Morvabon Pharmed Co.:  
Head Office: Unit 154, 7nd floor,7 block,  
Borjssaz Bd, Azadi Street, Tehran, Province, Iran  
Tel: 0098 21 6658 1838  
Fax: 0098 21 6690 8819  
Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,  
20thkm of Damavand Rd., Tehran, Iran  
Tel: 0098 21  
Fax: 0098 21

Production  
License No.  
21185230  
IMD: 94536813

- ISO13485 : 2016 , US cert, IAS, USA
- ISO9001: 2015, Tuv certification, Germany
- ISO14001: 2015, Tuv certification, Germany
- ISO18001: 2007, Tuv certification, Germany
- ISO10002: 2014, Tuv certification, Germany
- ISO10004: 2012, Tuv certification, Germany