MORVABON

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 manufacturer

Pardis Morvabon Pharmed Co.:

Head Office: Unit 154, 7nd floor,7 block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1438

Fax: 0098 21 6690 8819

Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran

Tel: 0098 21 7625 1702

Fax: 0098 21 7648 1457

MEDICAL EQUIPMENT
DEPARTMENT
IMD: 3079351

ISO 13485: 2016

Cavi-Mor

Temporary filling material

moisture-curing, radio-opaque, filling material for temporary filling of dental cavities.

composition: metal sulfates, organic resin

Applications:

- ▶ Temporary fillings of Class I and II cavities.
- ▶ Temporary sealing in endodontics.

Advantage:

- ▶ Ideal and appropriate setting time.
- ▶ Proper adhesion to dentin.
- ▶ Ease of use.
- ▶ Radiopaque.
- ▶ Minimum microleakage.
- ▶ Ready to use.
- ▶ No adhesion to instrument.

Intended use:

- ▶ Dry the prepared cavity with a small cotton pellet.
- ▶ Transfer the material to the cavity using a suitable instrument and form the contour. In the case of occlusal Class I cavities let the patient bite down on the surface, and then form the contour.
- ▶ Ask the patient to close his/her mouth. The filling will harden in about 30 minutes by absorbing moisture. Recommend the patient to avoid pressure or abrasion for at least 1 hour.
- ▶ Close the container immediately after use to prevent paste from stiffening.

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- ▶ It's necessary to use rubber-dam while working with **Cavi-Mor**.
- ▶ Use eye protection, mask and gloves when using **Cavi-Mor**
- ▶ If swallowed: there should be plenty of milk immediately and, in the absence of milk, drink lots of water, And then go to the Emergency Medical Center

- ▶ In case of contact with eyes, rinse thoroughly with water for 15 minutes and in case of continued burning, see your doctor.
- ▶ In case of frequent contact with skin, it causes irritation and dry skin, which should be washed with plenty of water and soap.
- ▶ Use protective gloves when using this product.
- ▶ Avoid contact with oral soft tissue, eyes or skin. If accidental contact with soft tissues, eyes or skin occurs,
- ▶ immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only

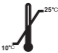
Warning:

- ▶ Only for dental use. Avoid contact with eyes and skin.
- ▶ Protect from the moisture and sunlight.
- ▶ Keep out of reach of children.
- ▶ Do not use after the expiration date.
- ▶ Store in the original container and kept at room temperature.
- ▶ Keep out of reach of children

contraindications:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

40 gram

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 Indicates the storage temperature range.

Produced by Morvabon Co.

www.morvabon.com

info@morvabon.com

Made in IRAN