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## Formo-Cresol Solution

Indicated as a disinfectant and devitalizing solution used as part of endodontic treatment.

**Composition:** Cresol, Formaldehyde, Alcohol

### Indications:

Formo-Cresol is a preparation used in treatment of putrescent or necrotic pulps. The formaldehyde acts as a germicide or fixing agent and renders all devitalized tissue inert and leathery, permitting easy removal. The Cresol acts as an efficient antiseptic and also reduces any irritation that could be caused by the formaldehyde.

### Intended to use:?

### Advantage:?

### Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Formo-Cresol Solution
- ▶ Use eye protection, mask and gloves when using Formo-Cresol Solution
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water. In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ DO NOT inject this product
- ▶ For dental use only
- ▶ This product should not be used with individuals with known sensitivities to formaldehyde or cresol.
- ▶ This product should not be used with individuals who are pregnant or suspected to be pregnant without a prior physician consultation.
- ▶ This product can cause severe burns of skin and mucosa. Take precautions protect skin, mucosa and eyes during use.

### warning:


- ▶ Do not use after the expiration date.
- ▶ Keep out of reach of children.
- ▶ Store between 10°-25°C.
- ▶ Protect from direct sunlight and freezing.
- ▶ Do not use after the expiration date.

### Direction for use:

- ▶ Intracanal disinfection in permanent teeth: Isolate the tooth with a rubber dam. Clean, de-bridge and instrument the canals. Rinse and dry the canal spaces. Moisten a small cotton pellet with formocresol. Blot or wring any excess liquid from the pellet. Place the cotton pellet in the pulp chamber and seal the access hole with a temporary filling material. Recall the patient after a minimum of 48 hours and remove the pellet. Complete the endodontic treatment and restore.
- ▶ Formocresol pulpotomy in primary teeth: Isolate the tooth with a rubber dam. Remove visible caries. Remove the coronal pulp with an excavator or bur leaving the radicular pulp. Achieve hemostasis. Moisten small cotton pellets with fomocresol. Blot or wring any excess liquid from the pellet. Place the pellets directly on the radicular pulp stumps for 5 minutes. Place a base or a core in the tooth and restore.

### Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance (37% phosphoric acid) and blue dyes.

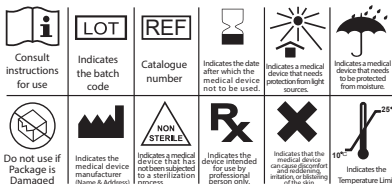## Formo-Cresol Solution

Indicated as a disinfectant and devitalizing solution used as part of endodontic treatment.

- ISO13485 : 2016 , US cert, IAS
- Iso 9001: 2015, Tuv certification
- Iso 14001: 2015, Tuv certification
- Iso 18001: 2007, Tuv certification
- Iso 10002: 2014, Tuv certification
- Iso 10004: 2012, Tuv certification

MEDICAL  
EQUIPMENT  
DEPARTMENT

manufacturer  
Pardis Morvabon Pharmed Co.;  
Head Office: Unit 154, 7nd floor, 7 block,  
Borjsoz Bd. Azadi Street, Tehran, Province, Iran  
Tel: 0098 21 6658 1438  
Fax: 0098 21 6690 8819  
Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,  
20thkm of Damavand Rd., Tehran, Iran  
Tel: 0098 21  
Fax: 0098 21