# MORVABON

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# Morva Etch (Acid Etchant Gel)

## Phosphoric Acid Blue Tint Gel 37%

**Morva Etch** gel ETCHANTS are phosphoric acid (37% H<sub>3</sub>PO<sub>4</sub>), polymer-thickened formulations intended for use on enamel and dentin. **Morva Etch** is a single-use, non-sterile product used for etching to promote adhesion of primer/bonding agent adhesives to tooth structure and restorative materials.

**Composition:** Phosphoric Acid, Gelling Agent, Polymeric Humectant, Blue dye.

**Morva Etch is available in two models:**

**Model 1:** contains a 50 ml luer lock syringe containing acid etch gel and closed with a protective cap, three non-sterile 3ml luer lock syringes without needles, three Prebent needle tip, 1/2", 20G yellow, one luer lock syringe cap, One connector, instructions and picture instructions for use in a cardboard box.

**Model 2:** contains 3×2.5 ml luer lock syringes containing acid etch gel and closed with a protective cap, three Prebent needle tip, 1/2", 20G yellow and the product instructions in a cardboard box.

The equipment required for 3 uses for the user is included in the product package. The user needs to provide a larger number of Prebent needle tip, 1/2" 20G yellow and connector and 3ml luer lock syringe (similar to the equipment in the product packaging) for the next use.

**Intended use:**

Etching gel is used in the total etch technique for the following indications:

- ▶ Etching dentin and enamel
- ▶ Cleaning agent on dental restorative material
- ▶ Composite restorations
- ▶ Luting of orthodontic appliances (e.g. brackets) and bonding of tooth jewelry
- ▶ Adhesive cementation of inlays (composite and all ceramic), crowns, bridges, veneers, adhesive bridges and posts and cores
- ▶ Pit and fissure sealing
- ▶ Splints

**Advantages:**

- ▶ The ideal viscosity of **Morva Etch** ensures effective application
- ▶ Rinses away cleanly and quickly
- ▶ The blue color residue is recognizable after rinsing
- ▶ The Prebent needle tip enables exact application
- ▶ The special protective cap prevent material from being wasted

**Precautions:**

- ▶ Carefully read and understand instructions before using.
- ▶ Re-cap, disinfect and wipe syringe with an intermediate level disinfectant between uses.
- ▶ Dispose of needle tip and empty Luer lock syringe properly.
- ▶ Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check.
- ▶ Use only recommended tips. To avoid cross contamination do not reuse tips.
- ▶ It's necessary to use rubber-dam while working with **Morva Etch**.
- ▶ Use eye protection, mask and gloves when using **Morva Etch**

- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only

**Warning:**

- ▶ Store at 15-25 °C (room temperature)
- ▶ Always replace original protective cap tightly after use
- ▶ Do not use after expiration date
- ▶ Keep out of reach of children

**Direction for use:**

After preparing the 3 ml syringes according to the cardboard pictorial instructions in **Model 1**, you can use 3ml syringe with the following directions for **Model 1** and **Model 2**:

- 1- Apply **MorvaEtch** on enamel for 20 seconds and subsequently on dentin and allow a reaction time of 10 seconds.
- 2- Thoroughly rinse off the etchant with water spray and dry the tooth surface with air. Avoid excessive dry.
- 3- Leave the surface slightly moist. The surface of enamel would be white chalky. If this appearance has not seen, continue etching for some more seconds.
- 4- If you want to apply hydrophilic bond, use cotton pellet for drying the enamel and dentin.
- 5- Drying have to be stopped before the white chalky surface has seen.

Note: In case of enamel impurities repeat the process for about 10 seconds.

**Contraindication:**

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance (37% phosphoric acid) and blue dyes.
- ▶ May be difficult to use in children, patients with special needs and the elderly as it may be hard to achieve good moisture control. If the humidity is controlled by the dentist while working, this product is easy to use and is not contraindicated.
- ▶ It's better not be used in patients where excellent moisture control is not possible.
- ▶ Do not use for those who are allergic to phosphoric acid and other ingredients of this product.

|   |   |   |  |   |   |   |   |
|---|---|---|--|---|---|---|---|
| <br>Consult instructions for use | <br>Indicates the batch code | <br>Catalogue number | <br>Indicates the date after which the medical device not to be used. | <br>Indicates a medical device that needs protection from light sources. | <br>Indicates a medical device that needs to be protected from moisture. | <br>Indicates the Temperature Limit<br>15°C 25°C | <br>CE Marking<br>0477 |
|---|---|---|--|---|---|---|---|

|   |   |   |   |  |  |   |
|---|---|---|---|--|--|---|
| <br>Do not use if Package is Damaged | <br>Indicates the medical device manufacturer (Name & Address) | <br>Indicates a medical device that has not been subjected to a sterilization process. | <br>Indicates the device intended for use by professional person only. | <br>Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin | <br>Indicates that the product has a chemical compound capable of metal corrosion or skin corrosion | <br>Indicates the authorized representative in the European Community/European Union |
|---|---|---|---|--|--|---|