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## Morva-Sealer

### Silver-Free Root Canal Sealing and Filling Material

**Morva-Sealer** is a material for sealing root canals. It has good tissue compatibility. This product consists of two components, powder and resin and it is suitable for all restorative techniques.

#### Compositions:

synthetic resin (oxirane base), accelerator, radiopacity agent (bismut base)

#### Intended use:

Root canal sealing material

#### Advantages:

- ▶ Resin-based, non-acrylic, eugenol-free sealer
- ▶ Excellent radiopacity
- ▶ Suitable for all obturation techniques

#### Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ During the setting reaction traces of formaldehyde are produced. Keep powder and resin container tightly closed.
- ▶ It's necessary to use rubber-dam while working with **Morva-Sealer**.
- ▶ Use eye protection, mask and gloves when using **Morva-Sealer**
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only

#### Warning:

- ▶ Store at 10-25°C
- ▶ Do not use after expiration date
- ▶ Keep out of reach of children


#### Instructions for use:

- ▶ Clean the root canal. Make sure that the canal is free of any traces of hydrogen peroxide solution. It can cause formation of bubbles. If sodium hypochlorite and hydrogen peroxide are used for canal irrigation, use sodium hypochlorite after hydrogen peroxide.

- ▶ Mix 2 or 3 volume units' powder with 1 volume unit resin on a glass slab with metal spatula.
- ▶ At the first use of the resin fluid tube, it is necessary to open the tube head with a thin needle. The amount of tube opening depends on the dentist's desired flow rate.
- ▶ Mix to a homogeneous consistency which breaks when lifted 1.5 to 2.5 cm above the glass slab.
- ▶ If a particularly fluid mix is required, the glass slab with the mixture may be carefully warmed over an alcohol flame.
- ▶ Apply the mixed cement into the tip of the Lentulo Spiral.
- ▶ If Morva-Sealer is used in combination with gutta-percha points or other cores and if there is risk of overfilling, use reamers to apply Morva-Sealer into the canal.
- ▶ It cannot be removed with conventional solvents. If Morva-Sealer is used with gutta-percha points, it can be removed using conventional techniques for the removal of gutta-percha.
- ▶ Spatulas, mixing slab and instruments should be cleaned immediately after use with alcohol or acetone.
- ▶ The setting time according to ISO 6876:2012 is between 9 to 15 hours at 37 °C.

#### Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance
- This product is not contraindicated for children, pregnant women, the elderly and certain diseases and can be used for all people except those who are allergic to the raw materials of this product (epoxy resin).

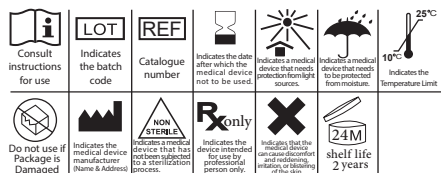Produced by Morvabon Co.  
www.morvabon.com  
info@morvabon.com  
Made in IRAN

### Morva-Sealer

Root Canal Sealing  
and Filling Material

- ISO13485 : 2016 , US cert, IAS, . ISO 18001: 2007, Tuv certification, USA Germany
- ISO 9001: 2015, Tuv certification, . ISO 10002: 2014, Tuv certification, Germany Germany
- ISO 14001: 2015, Tuv certification, . ISO 10004: 2012, Tuv certification, Germany Germany

Production  
License No.  
11231167  
IMD: 72296344

manufacturer

Pardis Morvabon Pharmed Co.;  
Head Office: Unit 154, 7nd floor, 7 block,  
Bojrsaz Bd, Azadi Street, Tehran, Province, Iran  
Tel: 0098 21 6658 1838  
Fax: 0098 21 6690 8819  
Factory: Unit 135, 136, 139, 2nd floor, Seraj Bd.,  
20thkm of Damavand Rd., Tehran, Iran  
Tel: 0098 21 7625 1702  
Fax: 0098 21 7648 1457