MORVABON

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manufacturer

Pardis Morvabon Pharmed Co.:

Head Office: Unit 154, 7nd floor, c block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1438

Fax: 0098 21 6690 8819

Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran

Tel: 0098 21 7625 1702

Fax: 0098 21 7648 1457

- ISO13485: 2016, US cert, IAS
- ISO 9001: 2015, TÜV certification
- ISO 14001: 2015, TÜV certification
- ISO 18001: 2007, TÜV certification
- ISO 10002: 2014, TÜV certification
- ISO 10004: 2012, TÜV certification

MEDICAL IFDA
EQUIPMENT
DEPARTMENT

Reg. No. at IRAN Medical
equipment department: 1855

ISO 13485: 2016

Oxy Block

Oxygen inhibitor glycerin gel

Oxyblock is a transparent and viscous gel based on glycerin, which leads to the formation of an oxygen inhibiting layer on the surface of the composite and resin materials and luting cements. This causes no disturbance in the polymerization process of the composite

Composition: Glycerin, gelling agent, water

Intended use:

- ▶ Protection against discoloration of composites in the vicinity of oxygen.

Advantages:

- ▶ This gel is easily washed off the surface of the composite.
- ▶ It does not contain harmful composite solvents.

Precautions:

- ▶ Before starting work, read the instructions carefully.
- ▶ It is necessary to use a rubber-dam while working with this product.
- ▶ When using this product, use protective glasses, mask and gloves.
- ▶ Do not use for patients who are highly allergic to the components of this product.
- ▶ In case of contact with soft tissues, eyes and skin, immediately wash the area with sufficient amount of water.
- ▶ Prevent the patient from swallowing the gel, if swallowed, drink plenty of water. Do not vomit and see a doctor.
- ▶ It should only be used for dental purposes and should not be used in similar cases.
- ▶ Do not inject this product

storage:

- ▶ Oxyblock Morvabon gel should be kept in its original packaging with the original lid.
- ▶ Replace the cap immediately after use.
- ▶ Keep away from moisture and direct sunlight.
- ▶ Do not use after the expiration date.
- ▶ Keep out of the reach of children.

Direction for Use:

- ▶ Place a quantity of oxyblock gel so that it covers the entire surface on the last layer of the composite and then cure it. After curing, wash the surface with water and finally do the finishing and polishing steps.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, elderly people and special patients and can be used by all people who are not allergic to the ingredients of this product.

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 Indicates the Temperature Limit