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## Prophy-Mor quick

Contain of micro & nano silica

**For Scrubbing curing, contains fluoride ions**

### Ingredients:

sodium fluoride, Soft and rough mineral silicate, pumice (micro and macro), glycerin, Nano and micro Silica, food colour additives.

### Intended use:

- ▶ Anti plaque, cleaner and containing Anticavity substances.
- ▶ Polishing suitable for teeth surface.
- ▶ Picked up and remove stains.

### Advantage:

- ▶ Easy usage by brushing the teeth
- ▶ Suitable adhesion for tooth placement during brushing
- ▶ Effective Scrubbing at the micro and Nano surface without damaging the gum and tooth
- ▶ Reducing microorganism of the mouth
- ▶ Helper as anti-decay (Containing fluoride ion)
- ▶ Having a major role for preventing gum of inflammations and gum of bleeding and gum disease caused by tooth plaque.

### Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with **Prophy-mor Quick**.
- ▶ Use eye protection, mask and gloves when using **Prophy-mor Quick**.
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water.
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice.
- ▶ DO NOT inject this product.
- ▶ For dental use only.

### warning:

- ▶ Keep out of reach of children.
- ▶ Do not use after the expiration date.
- ▶ Store in the original container and kept.
- ▶ at room temperature.
- ▶ Protect from direct sunlight.

### Direction to use:

- ▶ For using this paste, take a sufficient amount paste put on the tooth.
- ▶ The operation of the brushing should be done first with a little round and then with a high round.
- ▶ For interception contamination paste, close the door after use it, To restore the dough Removed avoid inside the dish.
- ▶ Throughout the procedure should be avoid excessive dilution and swallowing of the toth surface.
- ▶ Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.
- ▶ This dough does not need to be diluted with cold water or serum.

### Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

**Available packaging:** 150 grams

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device is for use on the surface of the skin.	 Indicates the Temperature Limit

Prophy-mor Quick paste

- ISO13485 : 2016 , US cert, IAS
- Iso 9001: 2015, Tuv certification
- Iso 14001: 2015, Tuv certification
- Iso 18001: 2007, Tuv certification
- Iso 10002: 2014, Tuv certification
- Iso 10004: 2012, Tuv certification

**Production License No.**  
**14776460**  
**IMD: 98923706**

manufacturer  
Pardis Morvabon Pharmed Co.:  
Head Office: Unit 154, 2nd floor, 7 block,  
Borazaj Bd, Azadi Street, Tehran, Province, Iran  
Tel: 0098 21 6658 1438  
Fax: 0098 21 6690 8819  
Factory: Unit 1 5,136,139, 2nd floor, Seraj Bd.,  
20thkm of Damavand Rd., Tehran, Iran  
Tel: 0098 21  
Fax: 0098 21