# ORVABON

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# Tempo Cem

## Temporary Dental Cement

**Tempo Cem** is a temporary cement that is very resistant. It is used for temporary bonding and placement of crowns, prosthetic pieces and bridges. Tolerating the pressure caused by chewing and at the same time being easy to move the restorative part is one of the characteristics of this cement. The time it takes after mixing equal amounts of two doughs is from 2 to 10 minutes, and changes in the ambient temperature and relative humidity change this time. Humidity can accelerate the setting. Each package contains a tube of base paste, a tube of hardener paste and a mixing plate. The content of base paste is zinc oxide weighing 60 grams and the content of gel hardener, eugenol and EBA weighing 30 grams.

### Contents of the package:

- ▶ Base paste (zinc oxide content): 55 grams
- ▶ Activator paste (containing eugenol and EBA): 18 grams
- ▶ Mixing pad: 1 pack

### Application and benefits:

- ▶ Suitable for temporary gluing of prosthetic pieces and bridges
- ▶ Temporary bonding and placement of crowns, prosthetic pieces and bridges
- ▶ Very resistant and high pressure tolerance
- ▶ Proper flow for better performance
- ▶ To place the crown
- ▶ Proper timing

### Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ Re-cap, disinfect and wipe TUBE with an intermediate level disinfectant between uses.
- ▶ It's necessary to use rubber-dam while working.
- ▶ Use eye protection, mask and gloves when using.
- ▶ Do not use for patients with hypersensitivity to the product's components.
- ▶ Avoid contact with oral soft tissue, eyes or skin.
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water.
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice.
- ▶ Do not inject this product.
- ▶ For dental use only.

## Warning:

- ▶ Tempo Cem contains eugenol. A small number of patients may experience an allergic reaction to eugenol.
- ▶ Always replace original protective cap tightly after use.
- ▶ Caution is recommended with these patients.
- ▶ Store at 15-25 °C (room temperature).
- ▶ Do not use after expiration date.
- ▶ Keep out of reach of children.

## Direction for use:

- ▶ **DIRECTIONS FOR USE Work Time and Set Time:** At the ideal mixing conditions of 21-25 °C and 40-60% relative humidity, the following work time and setting time from start of mixing are obtained: Tempo Cem : 1:30 < 10:00  
Mixing: Tube Extrude equal lengths of Base and Accelerator onto the mixing pad provided. The length to be extruded will depend on the size and type of restoration to be cemented. Replace caps tightly on tubes after use. Thoroughly mix the pastes for approximately 30 seconds.
- ▶ **Preparation and application:** Dry the prepared teeth and the surface of the restoration. Apply a thin layer of the mixed cement to the internal surfaces of the temporary restoration. Firmly seat the restoration in the mouth. After the material has set remove excess material with a scaler or other instrument.

**Made in Pardis Morvabon Pharmed Co.:**

**Head Office:** Unit 154, 7nd floor,7 block, Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1838 - Fax: 0098 21 6690 8819

[www.morvabon.com](http://www.morvabon.com) [info@morvabon.com](mailto:info@morvabon.com)

Made in IRAN

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.	 Indicates the Temperature Limit
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 The product is disposable	