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MEDICAL
EQUIPMENT
DEPARTMENT

IFDA

Reg. No. at IRAN Medical
equipment department: 1855

manufacturer

Pardis Morvabon Pharmed Co.:
Head Office: Unit 154, 7nd floor, c block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 14 38
Fax: 0098 21 6690 8819

Enamel Bond Morvabon

Hydrophobic Bond

Enamel Bond Morvabon is a hydrophobic, bonding resin specifically designed to bond to porcelain. Placed in a thin layer over a silane-repaired soil surface, Porcelain Bonding Resin acts as a wetting resin and allows the cement to spread in a more even layer.

Composition:

organic monomers, initiator

Intended use:

- ▶ Use as Enamel bond
- ▶ Use in composite veneers
- ▶ Composite lip repair
- ▶ Gluing fiber paste
- ▶ As a bond in the restoration of old composites
- ▶ use as porcelain resin bond

Advantages:

- ▶ Easy handling for composite shaping
- ▶ Without HEMA

Precautions:

- ▶ Carefully read and understand instruction before using.
- ▶ Avoid contact with oral soft tissue, eyes or skin . in case of consumption drink copious amount of water.do not induce vomiting. seek medical advice.
- ▶ if accidental contact with soft tissuse,eyes or skin occurs,immediately flush with plenty of water.
- ▶ Use eye protection, mask and gloves when using.
- ▶ do not inject this product.
- ▶ For dental use onlyWhen using dental adhesives, salivary contamination of the dentin is at risk. Bonding may result in reduced lifespan of the restoration. A rubber is recommended as a barrier for all direct restorations and other indications.

warning:

- ▶ Store at 5-25 °C.
- ▶ Keep away from moisture and direct sunlight.
- ▶ Close the lid of the product after use to prevent absorption.
- ▶ Do not use after expiration date.
- ▶ Keep out of reach of children.

Direction for use:

- ▶ First, etch the porcelain and then apply a layer of silane on the desired surface, then put a thin layer of the product and cure for 20 seconds, and then apply the desired restoration.

Contraindication:

- ▶ This product is not contraindicated for children , pregnant women ,the elderly and other specific diseases and can be used for all people who are not allergic to the active substance.

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 Indicates the Temperature Limit